Control β_2 -microglobulina
MonlabTest®

Turbidimetría

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β_2 -microglobulin Control
MonlabTest®

Turbidimetry

Quantitative determination of β_2 -microglobulin

Only for professional *in vitro* diagnostic use.
Store at 2-8°C.

PRODUCT CHARACTERISTICS

The control is a lyophilized human serum containing β_2 -m used to evaluate the precision and accuracy of β_2 -m determinations in human serum by turbidimetry method.

REAGENTS

β_2 -m Control	Human serum. With a low concentration level of β_2 -m. The β_2 -m concentration is stated on the vial label.
----------------------	--

PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However, handle cautiously as potentially infectious.

CALIBRATION

The concentration value of the control is calibrated against the 1st International Standard ERM-DA470k/IFCC.

PREPARATION

Reconstitute (→) with 2.0 mL of distilled water. Mix thoroughly, avoiding foam forming. Bring at room temperature for about 10 minutes before use.

STORAGE AND STABILITY

The control is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

After reconstitution, stable for 10 days at 2-8°C or 6 months at -20°C.

PROCEDURE

To be used in turbidimetric assays.

PRESENTATION

MO-165054

1 x 2 mL