<p><b>Control Ferritina</b>  <b>MonlabTest®</b>                  Turbidimetría</p>	 
--	------

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**Quantitative determination of Ferritin**

Only for professional *in vitro* diagnostic use.  
 Store at 2-8°C.

**PRODUCT CHARACTERISTICS**

The control is a lyophilised serum from human origin containing ferritin used to evaluate the precision and accuracy of ferritin determinations by turbidimetry method.

**REAGENTS**

Ferritin Control	Human serum. With a low concentration level of ferritin. The value of ferritin is stated on the vial label.
------------------	---

**PRECAUTIONS**

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However, handle cautiously as potentially infectious.

**CALIBRATION**

The concentration value of the control has been standardized against the 2<sup>nd</sup> International Ferritin Standard (80/578) from WHO (1992).

**PREPARATION**

Reconstitute (→) with 2.0 mL of distilled water. Mix thoroughly, avoiding foam forming. Bring at room temperature for about 10 minutes before use.

**STORAGE AND STABILITY**

The control is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

After reconstitution, stable for 1 month at 2-8°C or 3 months at -20°C

**PROCEDURE**

To be used in turbidimetric assays.

**PRESENTATION**

MO-165056	1 x 2 mL
-----------	----------